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**INSTRUCTIONS FOR USE (Canada)** – This leaflet contains important product use and safety information. Please read carefully and retain for future reference. Contact lenses are made of different materials and dimensions and should always be fitted by an eye care professional. It is essential that contact lens wearers see their eye care professional regularly, and follow their directions and all labelling instructions for proper use of contact lenses and lens care products.

**PRODUCT NAME(s)**  
ClearView Locator Tint

**DESCRIPTION**  
The lenses are available in a spherical lens design and in a range of dimensions and refractive powers. The eye care professional will prescribe the appropriate lens design that's needed to correct vision in your right and left eye. When placed on the cornea, the lenses act to correct vision by refocusing light rays on the retina.

The lens material is 62% polymacon polymer and 38% water by weight when fully hydrated in 0.9% sodium chloride solution. The polymacon lens material is a hydrophilic polymer of 2-hydroxyethyl methacrylate crosslinked with ethylene glycol dimethacrylate. ClearView Locator Tint lenses contain a light blue color (VAT Blue 6) to make them easier to see when handling.

Each lens is supplied in a glass vial containing 0.9% sterile sodium chloride solution. The glass vial is marked with the base curve, diameter, power, lot number and expiration date. Do not use if the tamper evident seal has been broken.

Note: THE PACKAGING OF THIS PRODUCT CONTAINS DRY, NATURAL RUBBER.

The Health Canada Device Identifier for ClearView Locator Tint contact lenses is 19930515.

**INTENDED USE**

Use the lenses to improve the vision of non-diseased eyes. Spherical lenses are available for persons who are nearsighted (myopia) or farsighted (hyperopia) and who may exhibit up to 2.00D of astigmatism that does not interfere with visual acuity.

**RECOMMENDED WEAR AND REPLACEMENT SCHEDULE**

- In the interest of maintaining the health and safety of your eyes, the wear and replacement schedule should be determined by your eye care professional.
- ClearView Locator Tint lenses are for daily wear only (less than 24-hours per day, while awake). Lenses should be removed each day prior to sleep, with cleaning and disinfection prior to reinsertion on the eye.
- To start lens wear, the eye care professional may prescribe an hourly schedule of increasing wear time per day over a period of several days or weeks until the wearer has successfully adapted to contact lens wear.
- Normal daily wear of lenses assumes a minimum of 6 hours of non lens wear per 24 hour period, however individual wearing schedule will vary. The eye care professional should determine how many hours per day lenses may be worn.

CIBA VISION recommends that ClearView Locator Tint lenses be replaced once a year. Based on this recommendation, it is up to your eye care professional to determine the actual wear and replacement schedule that is best for you.

**WHEN NOT TO WEAR LENSES**

**DO NOT USE ClearView Locator Tint contact lenses when any of the following exists:**

- Allergy, inflammation, infection or irritation on or around the eye or eyelids.
- The use of some medications, including eye medications. Always consult your eye care professional before using any medicines in your eyes.
- Have a systemic disease that may be affected by or impact contact lens wear.
- Have certain types of allergic conditions.
- Inadequate tear film (dry eyes).
- Water sports without use of goggles.
- If eyes become red or irritated.

Consult with the eye care professional specifically regarding these or other conditions.

**IMPORTANT THINGS FOR WEARERS TO REMEMBER**

- DO NOT WEAR ClearView Locator Tint lenses while sleeping. Overnight wear of contact lenses has been shown to increase the risk of serious ocular complications and smoking increases these risks.
- It is essential that contact lens wearers see their eye care professional regularly. It is recommended contact lens wearers see their eye care professional twice each year or as recommended by the eye care professional.
- Do not use lenses past the expiration date and only use fresh, unexpired lens care products.
- Cosmetics, lotions, soaps, creams or deodorants can all cause irritation if they come in contact with your lenses and should be used carefully.
- Remove lenses when exposed to harmful or irritating sprays, vapors and fumes.
- Consult your eye care professional about wearing lenses during sporting activities, including swimming.
- Inform an employer that you wear contact lenses, especially if your job involves using eye protection equipment.

**POSSIBLE PROBLEMS**

While wearing contact lenses the eyes should look well, feel comfortable and vision should be clear.

Although contact lenses provide many benefits to the wearer, it is possible that problems can occur and may be first noticed as one or more of the following conditions:

- Feeling of something in the eye
- Uncomfortable lenses
- Eye redness
- Sensitivity to light
- Burning, stinging, itching or watery eyes
- Reduced sharpness of vision
- Rainbows or halos around lights
- Increased eye secretions
- Severe or persistent dry eyes

**WHAT TO DO IF A PROBLEM OCCURS**

If any of the above symptoms occur:

**• IMMEDIATELY REMOVE THE LENSES.**

- If the discomfort or problem stops, then look closely at the lens(es):
  - If the lens(es) is in any way damaged, DO NOT put the lens(es) back on the eye. Replace with a new lens or consult the eye care professional.
  - If the lens(es) have dirt, an eye lash or other foreign body on it, or the problem stops and the lenses appear undamaged, thoroughly clean, rinse, and disinfect prior to reinsertion.
- If the discomfort or problem continues after removing lens(es) or upon reinsertion, IMMEDIATELY remove the lens(es) and contact the eye care professional for identification of the problem and prompt treatment to avoid serious eye damage.

A serious condition such as infection, corneal ulcer (ulcerative keratitis), corneal vascularization, or iritis may be present. These conditions could progress rapidly and may lead to permanent loss of vision. Less serious reactions such as abrasions, infiltrates, and bacterial conjunctivitis must be managed and treated carefully to avoid more serious complications. Professional identification of the problem and prompt treatment are necessary to avoid serious ocular complications.

**HANDLING LENSES**

The eye care professional should provide wearers with detailed contact lens wear, lens care, insertion and removal instructions.

- Always wash and dry hands before handling. Ensure the lens is right side out and that the correct lens for each eye is available. Inspect lenses prior to insertion. Do not insert lenses if damaged.
- To insert lenses:
  - Place a lens on the tip of your clean and dry right or left index finger, place the middle finger of the same hand close to your lower eyelashes and pull down the lower eyelid.
  - Use the fingers of the other hand to lift the upper eyelid.
  - Place the lens directly on the eye (cornea) and gently roll your finger away from the lens.
  - Look down and slowly remove your hand, releasing the lower lid.
  - Look straight ahead and slowly remove your other hand, releasing the upper lid.
  - Blink gently.
- To remove lenses:
  - Make sure hands are completely dry.
  - Blink fully several times.
  - While looking up, slide the lens down onto the white part of the eye.
  - Remove the lens by pinching gently between the thumb and forefinger.
- If a lens decenters on the eye, close eye and gently massage the eyelid to return the lens to the central position. If the problem persists, consult the eye care professional.
- Never use tweezers, sharp objects or fingernails to move, insert or remove lenses.

**TAKING CARE OF LENSES**

- Good hygiene and correct care of lenses are essential.
- DISINFECT lenses each time they are removed to destroy harmful germs and to ensure safe and comfortable contact lens wear. Use a chemical, not heat, disinfection system.
- A separate CLEANING & RINSING step may be recommended by the eye care professional to remove contaminants from the lens surface. In addition, lenses may also require periodic enzymatic cleaning to remove protein build-up.
- Remove one lens first (always the same lens first to avoid mix-ups) and place the lens into the correct chamber of the lens storage case. Repeat this procedure for the second lens.
- The eye care professional should recommend an appropriate lens care system. CIBA VISION recommends the following for use with ClearView Locator Tint lenses:
  - Hydrogen peroxide disinfecting system – Clear Care®
  - Chemical cleaning & disinfecting solutions – SOLOCARE AQUA® All-In-One Solution
  - Cleaner – ADFLOW® Extra Strength Daily Cleaner
  - Protein Removal – Unizyme® Enzymatic Cleaner
  - Lubricating & wetting drops – Aquify® Long-Lasting Comfort Drops
- To store lenses, clean, disinfect, neutralize, rinse, and leave them in the closed/unopened lens storage case until ready to wear. If the lenses have been stored in the unopened lens storage case for more than 24 hours, repeat the cleaning, disinfecting, neutralizing and rinsing procedures using fresh solutions prior to wearing the lenses.
- Contact lens cases require proper use, cleaning and replacement at regular intervals. To prevent contamination and avoid serious eye injury, follow the manufacturer's instructions for use and care of the lens case and replace the lens case according to the manufacturer's instructions, or as recommended by the eye care professional.

**Important notes:**

- DO NOT USE saliva, tap water, distilled water, or homemade saline solutions to care for or when handling your lenses. The use of tap and distilled water has been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure.
- DO NOT USE products designed only for hard or rigid gas permeable lenses.
- NEVER USE HEAT with lens care solutions or to disinfect lenses.
- Refer to the package inserts of each individual lens care product for important safety information and complete directions.

**CARE FOR A DEHYDRATED LENS**

If a ClearView Locator Tint lens is exposed to air while off the eye, it may become dry and brittle. To rehydrate the lens:

- Handle the lens carefully.
- Place the lens in its storage case and soak the lens in a recommended rinsing and storing solution for at least an hour until it returns to a soft state.
- Clean and disinfect the rehydrated lens using a recommended lens care system.
- If, after soaking, the lens does not become soft, DO NOT USE THE LENS, but contact your eye care professional.

**CARE FOR A STICKING LENS**

A lens must move freely on the eye for the continued health of the eye. If the lens sticks (stops moving) on the eye, apply 2-3 drops of a sterile saline solution recommended for use in eyes. Wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues after several minutes, immediately consult the eye care professional.

**GENERAL EMERGENCIES**

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes:

- FLUSH EYES IMMEDIATELY WITH FRESH SALINE SOLUTION OR TAP WATER.
- REMOVE AND DISCARD THE LENS, AND IMMEDIATELY CONTACT YOUR EYE CARE PROFESSIONAL OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.

CIBA VISION Corporation  
11460 Johns Creek Parkway  
Duluth, Georgia 30097 USA  
www.cibavision.com

CIBA VISION Canada, Inc.  
2150 Torquay Mews,  
Mississauga, Ontario  
L5N 2M6 Canada

Date: November 2008  
Printed in: Canada