

Aquify®

**LONG-LASTING
COMFORT DROPS**

PLEASE READ CAREFULLY AND KEEP THIS PACKAGE INSERT FOR FUTURE USE.

DESCRIPTION:

AQuify® Long-Lasting Comfort Drops is a sterile solution containing sodium hyaluronate, sodium chloride, sodium phosphate and sodium perborate stabilized with phosphoric acid as a preservative. For use with Soft and Rigid Gas Permeable (RGP) contact lenses.

Fresh Comfort with Every Blink

- Special blink-activated ingredient behaves like natural tears.
- Provides unmatched lubrication and moisture retention.
- Patented disappearing preservative helps prevent irritation.

INDICATIONS:

Use AQuify Long-Lasting Comfort Drops to moisten, cushion, refresh, and provide temporary relief from dryness and irritation associated with contact lens wear.

ACTIONS:

- AQuify Long-Lasting Comfort Drops is especially formulated to lubricate and moisten your lenses, providing relief from dryness and irritation by re-wetting the lens.
- During lens wear, Aquify Long Lasting Comfort Drops helps remove and prevent protein build-up to ensure that lenses stay fresh and comfortable.

DIRECTIONS FOR USE:

Place one or two drops in each eye and blink several times.

To help prevent protein build-up, place 2 drops in each eye up to 3-4 times daily.

CONTRAINDICATIONS:

There are no known contraindications this product, however if you are allergic to any ingredient in this solution, do not use.

WARNINGS:

PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. Follow your eye care practitioner's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Prompt diagnosis and treatment are essential; therefore, if you experience eye discomfort, pain, unusual eye secretions, excessive tearing, sensitivity to light, vision changes, or redness of the eye, immediately remove your lenses and promptly contact your eye care professional.

- To avoid contaminating, DO NOT touch tip of container to any surface. Replace cap after using.
- To avoid contaminating, DO NOT transfer to other bottles or containers.
- All contact lens wearers should see their eye care practitioner as directed. If your lenses are extended wear, your eye care professional may prescribe more frequent visits.
- Studies have shown that smoking increases the risk of ulcerative keratitis for contact lens users.

PRECAUTIONS:

General Precautions:

- Never reuse this solution.
- Keep the bottle tightly closed when not in use.
- Use before the expiration date marked on the carton and label.

- Keep out of the reach of children.
- Store at room temperature.

ADVERSE REACTIONS

(Problems and what to do):

The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above **IMMEDIATELY REMOVE YOUR LENSES.**

- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, **DO NOT** put the lens back on your eye. Place the lens in the storage case and contact your eye care practitioner.
- If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, thoroughly clean, rinse and disinfect the lens, then reinsert it.
- If the problem continues, **IMMEDIATELY** remove the lens and consult your eye care practitioner.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

All adverse reactions observed while using this product should be reported to:

CIBA Vision Corporation, Consumer Relations

Duluth, GA 30097

Or call us toll free at 1-800-875-3001

HOW SUPPLIED:

This product is supplied sterile in 0.066 fl. oz. (2 ml) and 0.33 fl. oz. (10 ml) bottles. Each bottle is marked with a lot number and an expiration date.

©2002 CIBA Vision Corporation

Made in Canada

Manufactured for CIBA Vision Corporation

Duluth, GA 30097 USA

December 2004

S7424A

**CIBA
Vision®**
A Novartis Company

